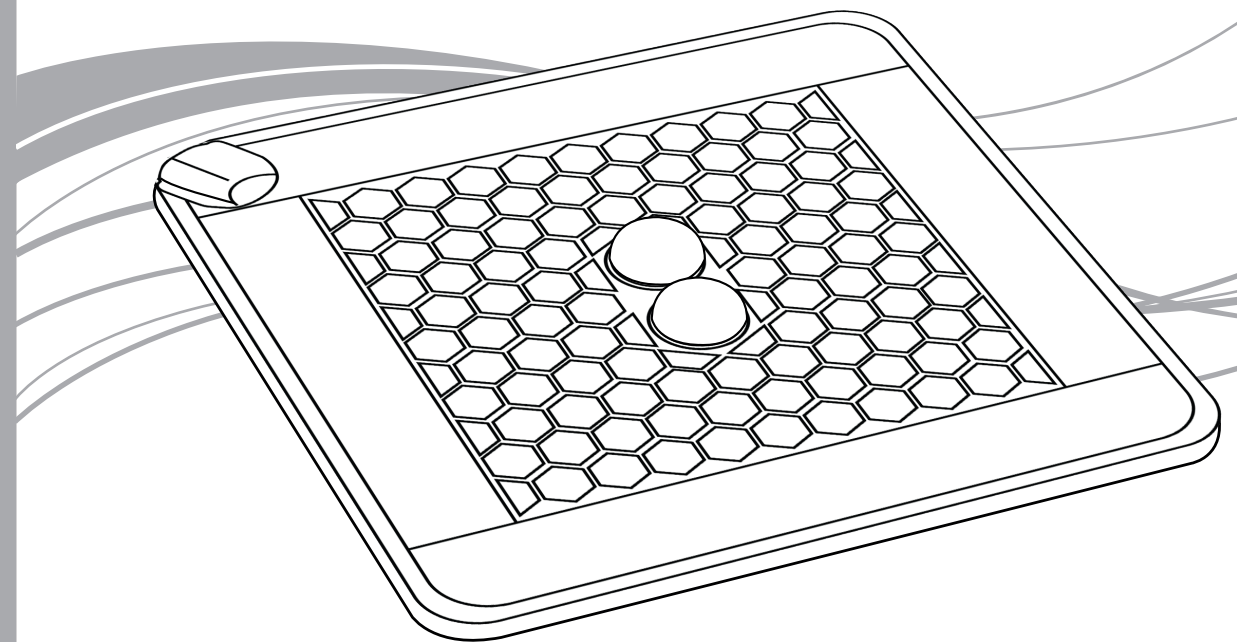




User Manual

CGM EMT-CS1302



Head Office
/ Factory

10, Jeongja 1-gil, Seonggeo-eup, Seobuk-gu,
Cheonan-si, Chungcheongnam-do, 331-831, Korea
TEL : (82)41-523-2550
FAX : (82)41-523-2560

Homepage

<http://www.ceragem.net>



Important safety issues

- Please read this manual thoroughly before using the product to ensure proper use.
- The cautions contained in this manual describe crucial matters on your safety.
Be sure to follow them at all times.
- Do not connect the control unit to the other appliance.
- Appliance must be returned to the manufacturer or distributor if the fuse has ruptured.
- This appliance is not intended for medical use in hospitals.

1. Do not dismantle, repair, or modify this product arbitrarily.
 - Contact our service center if the product needs repair. Unauthorized repairs pose risk of personal injuries such as fire or electric shock.
2. Do not pull the power cord or touch the plug with wet hands.
(Be sure to hold the plug and pull to unplug.)
 - This poses risk of electric shock or fire.
3. Do not use damaged power cord or plug or loose socket.
 - This poses risk of electric shock or fire.
4. Do not use voltage other than the rated voltage.
 - This poses risk of electric shock or fire.
5. Children or disabled persons must use the product in the presence of a guardian.
 - Failure to do so poses risk of burns or injuries.
6. Never go out or move to another place while the cushion is in operation.
(Be sure to turn power off first when you move.)
7. Do not put a blanket, other flammable materials, or anything heavy on the cushion.
 - This may cause fire.
8. Never store this product folded multiple times or use it in folded state.
 - This may cause product failure or fire.
9. Disconnect the plug from the socket when the product is not in use.
10. Do not wet the main body and controller.
(If water permeates, there is risk of overheating and electric shock.
In such case, stop using the product and pull the power plug immediately.)
 - Do not use the product until it is completely dried. In case of abnormality, contact our service center.
11. Do not put the product in an unsafe place.
 - If you put the product on an uneven ground, it can fall or drop
(may cause product failure).

- Do not use the product on the cushion or inside the blanket. (This may cause malfunction.)
- Do not give a shock such as throwing or dropping the controller.
- Do not use on bare skin.

Contents

>> Important safety issues

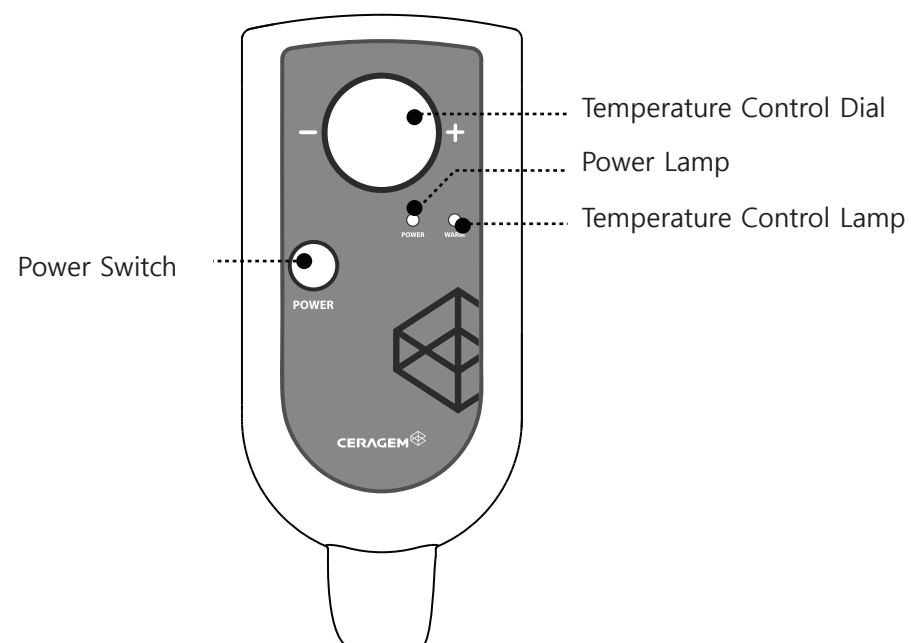
| | |
|--|---|
| 1. The name of each part ----- | 2 |
| 2. Using the product ----- | 3 |
| 3. Management techniques and Product Specifications ----- | 4 |
| 4. Product Warranty ----- | 5 |



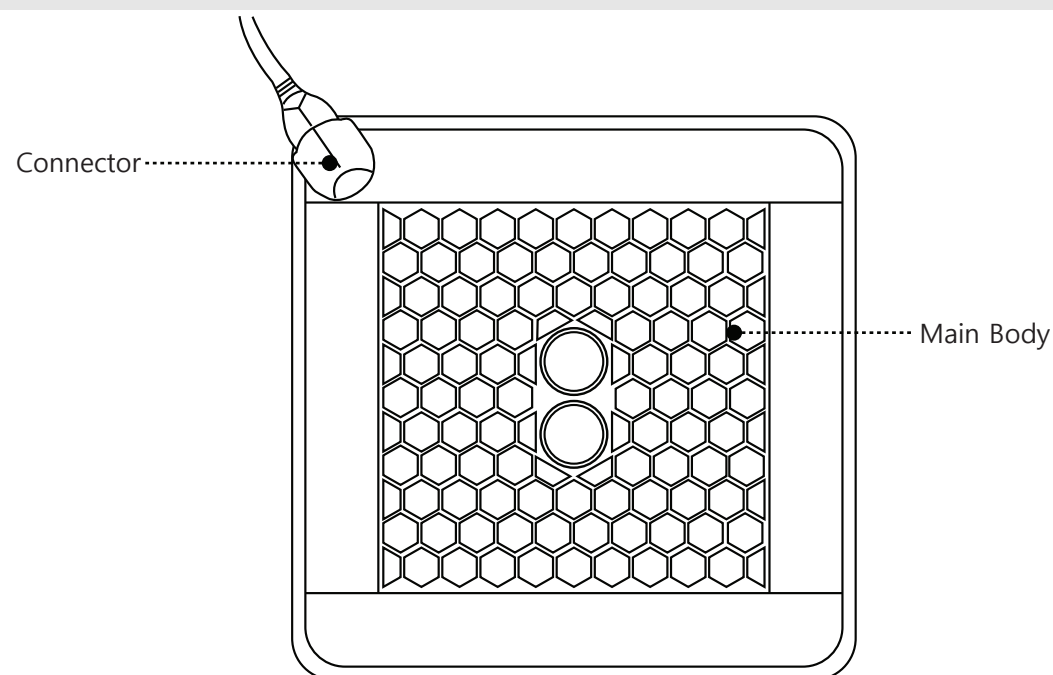


1. The name of each part

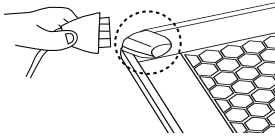
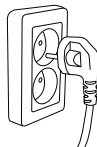
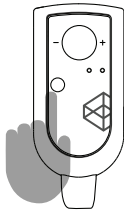
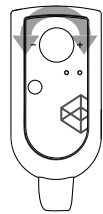
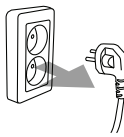
Controller



Cushion



2. Using the product

| | |
|--|---|
| 1. Connect the connector  | 1. Connect the temperature controller connector to that of the mat's main body. - Both grooves must match for connection. |
| 2. Connect the power cord  | 1. Connect the power cord to the socket. - This product is used only for 220V-240V~ Before connecting the cord, be sure to check ※ Do not use loose socket or damaged power code. |
| 3. Use the temperature controller  | 1. Connect the power cord and press the power switch on the controller to turn power on. - Then power lamp turns on. 2. When the setting temperature is low, temperature control lamp turns on. And once the temperature reaches the setting temperature, it turns off automatically. 3. Press the power switch once again to turn power off. 4. After 12 hours, temperature controller turns off automatically after 12 hours. |
| 4. How to control the temperature  | 1. After pressing the power switch, fix the temperature control dial to the desired temperature. (Temperature - : 30°C / + : 65°C) |
| 5. When the product is not in use  | 1. Press the power switch once again to turn power off. 2. Be sure to disconnect the power cord from the socket when the product is not in use. |



Caution

Extended period of use at high temperature poses risk of burns.
Be sure to set the dial to "-" when using the product for an extended period.



3. Management techniques and Product Specifications

Management techniques and storage methods

- 1) Unplug the power cord from the socket before cleaning the product.
- 2) Do not wash the main body and controller. (risk of electric shock).
- 3) Avoid direct sunlight or sharp change of temperature and humid places.
- 4) Wait until the product cools down and avoid creases when storing it.
- 5) Check often for any damage in the product. Stop using the product and contact the A/S center in case of abnormality or misuse.

Self-check – Before Contacting Us for A/S

| |
|---|
| Power is not turned on. - Is power not turned on? Check the main power. - Is the power cord connected well? |
| The cushion is not heated. - Is the temperature controller connected to the cushion's main body? - Is the temperature controller dial properly set? |
| The Controller is heated. - There is slight heat generated since the heat shield is installed inside the temperature controller. |

Product Specifications

| | |
|---------------------------|-------------------------------|
| Article | Ceratonic Cushion(Ceramic C1) |
| Model | CGM EMT-CS1302 |
| Voltage | 220V-240V~, 50/60Hz |
| Consumption Power | 60W ±10% |
| Temperature Available | 30°C~65°C |
| Measurement | 470mm x 470mm x 35mm(WxLxH) |
| Weight(Except Controller) | 1.9Kg |

- ※ The design and product specification may be changed without prior notice for purposes of quality improvement.
- ※ The contents of the user manual may be changed according to the company's circumstances without prior notice.



4. Product Warranty

Warranty

Article : Ceratonic Cushion(Ceramic C1), Model : CGM EMT-CS1302

CERAGEM provides warranty for the product described above according to the regulations of the Customer Protection Law.

| | | | |
|----------------------------|----------------|-------------|--|
| Manufacturing No. | | | |
| Year/Month/Day of Purchase | | | |
| Free Warranty Period | | | |
| Information of Customer | Name | Contact No. | |
| | Address | | |
| Place of Purchase | Name of Dealer | Contact No. | |

In case you do not have a product warranty, or you did not fill out the required fields, you will be either entitled to free service or charged for A/S in accordance with the additional standard for the quality warranty period.

If you experience any failure in your product, be sure to contact the dealer where you made your purchase. In case your dealer cannot handle the problem, contact the customer support center for service.

Standard for Customer Service Support

1. The free warranty period for this product is one year.
2. If you experience any spontaneous failure under normal operation during the 1-year free warranty period from the date of purchase, you are entitled to free service. Under the following cases, however, you will be charged for the service even if the free warranty period is still valid:
 - Failure and damage due to careless handling
 - Failure and damage due to natural disasters
 - Failure due to non-observance of the usage instructions and cautions described in the user's manual
 - Failure due to unauthorized repair or modification by the customer
 - Consumables whose service life has ended
 - Failure occurring in an environment outside the range stated in the product warranty
 - Other abnormal operations due to connection to products of other manufacturers
3. This warranty is valid only domestically. Since it is not reissued, take care not to lose it.

This warranty guarantees free service based on the duration and condition above. Therefore, it does not restrict the customer's legal rights. For questions on repair beyond the warranty period, contact the dealer where you purchased the product or the customer service center.

CERAGEM CO., LTD.
10, Jeongja 1-gil, Seonggeo-eup, Seobuk-gu,
Cheonan-si, Chungcheongnam-do, 331-831, Korea
Tel : (82)41-523-2550 Fax : (82)41-523-2560
Homepage : <http://www.ceragem.net>